DE Gehhilfe

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EN Walking Aid

To determine the correct size (see image), please measure the circumference directly behind elbow (1) and the folds of the knee (2) when the dog is standing. If the dog has a thick coat, the tape measure needs to be tight.

Acquaint your dog with the walking aid gently and make sure you do not overstress the dog. In case an operation is planned, it is advisable to try out the walking aid beforehand so that your dog is relaxed when being acquainted with the new aid.

When putting on or while using the walking aid, pay attention that no body parts are constricted or pinched particularly with male dogs. Adjust the length of the handles so that you can comfortably stand upright next to your dog to avoid straining your own back. Thanks to the integrated D ring, the walking aid can be combined with a harness to provide a secure hold. Do not make your dog wear the walking aid longer than necessary. If you are unsure whether this item is suitable for use with your dog, please consult your veterinarian.

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